

RAPID DIAGNOSTICS

osom[®]

Mono Test

TO AID IN THE DETECTION OF MONONUCLEOSIS



www.osomtests.com



FEATURES:

- ✓ **ACCURATE;** >99% sensitivity;
96% specificity
- ✓ **FAST;** Results in 5 minutes
- ✓ **SAMPLE;** Whole blood, serum or
plasma
- ✓ **NO AGE LIMITATIONS**
- ✓ **QC INSIDE;** Includes 2 additional tests
for external QC
- ✓ **TRAINING;** Online training and
certification; www.osomtraining.com
- ✓ **CUSTOM CARE;** Technical support
staffed with certified medical
technologists

RAPID DIAGNOSTICS

www.osomtests.com

osom[®] Mono Test

Sekisui Diagnostics, LLC (AMERICAS)
4 Hartwell Place Lexington, MA 02421
Phone: 800 332 1042
Fax: 800 762 6311
Email: osomrapidtests@sekisui-dx.com
ISO 13485 Certified

Sekisui Diagnostics (UK) Limited (INTERNATIONAL)
50 Gibson Drive, Kings Hill, West Malling, Kent, ME19 4AF, UK
Phone: +44 (0) 1732 220022 Fax: +44 (0) 1732 220024
Email: info@sekisui-dx.com
ISO 9001 Certified, ISO 13485 Certified

TEST PROCEDURE / RESULTS



For serum, plasma or whole blood from collection tubes, add 1 drop to test tube from pipette.

For fingertip blood, fill capillary tube and dispense blood into test tube.




Add 1 drop of diluent to test tube. Mix.

Place the test stick into the treated sample.



Read results at
5 MINUTES.

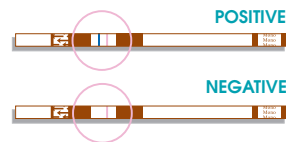

DESCRIPTION

OSOM[®] MONO TEST

CONFIGURATION

25 TESTS + 2 TESTS FOR QC

CATALOG

145 (CE MARKED)

STORAGE

ROOM TEMP

CPT CODE

86308QW (WHOLE BLOOD)
86308 (SERUM, PLASMA)

